ES

Descrpción:

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EN

Description:

Alpha-Ease® desensitizer is a simple one-step product for treating and preventing dentinal hypersensitivity. It works quickly and effectively to seal the dentinal tubules and inhibit bacterial growth.

Indications for Use:

- Under any indirect resin-bonded inlay, onlay, crown or veneer.
- Under amalgam and composite restorations.
- In conjunction with other dentin bonding adhesives.

Contraindications:

Alpha-Ease® is contraindicated in patients with a history of sensitivity to HEMA or glutaraldehyde. Do not use **Alpha-Ease**® if the pain is due to pulpitis, if the pulp is exposed, or if the dentine adjacent to the pulp is perforated.

Warnings:

- Avoid contact of **Alpha-Ease**® with soft tissue, eyes, and skin. In case of contact with soft tissue, eyes or skin immediately flush with water. If accidental eye contact occurs, immediately flush with water and seek medical advice. Extended contact with soft tissue, eyes, and skin can result in serious damage.
- **Alpha-Ease**® is harmful by inhalation, is irritating to the respiratory system and may cause sensitization.

General Precautions:

- U.S. Federal law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.
- Always replace the cap after each use.
- Proper personal protection equipment must be used to prevent desensitizer from coming in contact with soft tissue, eyes or skin.
- Irritation of the pulp may be observed in deep cavities without pulp protection. Suitable cavity linings are required to protect the pulp.

Adverse Reactions:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for use:

1. Isolate the tooth using a rubber dam.
2. Prepare and clean dentin surfaces. It is not necessary that the dentin surface is completely dry, a slight moist surface is acceptable.
3. Apply **Alpha-Ease**® with a brush or any other suitable instrument. A gentle rubbing motion for 10 seconds will assist the delivery of the desensitizer into the dentinal tubules.
4. Allow desensitizer to remain for 30 to 60 seconds.
5. Dry the surface carefully by applying a stream of compressed air until the fluid film has disappeared and the surface is no longer shiny.
6. Repeat application if sensitivity recurs or persists.

Storage:

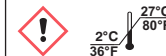
- Refrigeration is recommended. Store material between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.

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