ES

**ALPHA-PRO® PASTA PROPHY**

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EN

**ALPHA-PRO® PROPHY PASTE**

**Alpha-Pro®** Propy Paste is a blend of polishing and cleaning agents designed for professional use during standard dental/orthodontic hygiene procedures. **Alpha-Pro®** Propy Paste is packaged in single unit dose cups or bulk jars.

**Indications for use:**

For cleaning and polishing procedures as part of a professionally administered dental/orthodontic prophylaxis treatment. If product contains fluoride, it is not considered a substitute for other forms of fluoride treatment.

**Instructions for use:**

- 1a. If using a unit dose cup, using a propy angle, fill the cup by pressing the face of the cup into the paste while running the angle at a low speed.
- 1b. If using a bulk jar, remove a sufficient amount of paste to complete one procedure and plate in a clean dappen dish or other suitable container. Immediately close lid after use to avoid moisture loss. Using a propy angle, fill the cup by pressing the face of the cup into the paste while running the angle at a low speed.
2. Maintain a quick and light touch so the edges of the cup flair slightly while continuing moving from the tooth. Sufficient paste should be maintained on the cup to avoid overheating of the tooth surface.
3. Evacuation of saliva is recommended to avoid dilution and swallowing of the paste.
4. Rinse full mouth with water spray.

**Storage:**

Do not store at temperature above 25°C/77°F. Keep out of direct sunlight. See outer packaging for expiration date.

**Warnings:**

- Propy paste should be expectorated and not swallowed.
- Product may contain Fluoride, nausea can result from excessive ingestion of fluoride. See outer label for details.
- Keep out of reach of children.

**Caution:**

U.S. Federal Law restricts this device to sale by or on the order of a dentist.

DENTAL  
TECHNOLOGIES

ALPHA-PRO  
PROPHY-PASTE

DE PROPHYLAXISPASTE

ES PASTA PROPHY

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IT PASTA PROFILASSI

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NL PROFYLAXEPASTA

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TR PROFILAKSI CILA PASTASI

UZ PROFILAKTIKA PASTASI

20°C  
68°F

25°C  
77°F

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a dentist.

PM-5728 Rev000

DENTAL  
TECHNOLOGIES

6901 N. HAMLIN AVENUE  
LINCOLNWOOD, ILLINOIS 60712 USA

www.dentaltech.com

EMERGO EUROPE

Prinsessegracht 20

2514 AP, The Hague

The Netherlands